14  
mm28  
mm

## EN) Vaginal Veil Collector V-Veil UP2<sup>®</sup>

Instructions  
Please read and keep this leaflet

**WHAT IS "V-Veil UP2" AND WHEN CAN IT BE USED?**  
The "V-Veil UP2" is a Class I, non-sterile, UVC disinfected medical device used for the collection of secretions, various fluids and substances from the vaginal cavity and the cervix for diagnostic laboratory analysis. This device may be used during bleeding, pregnancy or following toxic shock syndrome only on the advice of a medical professional. "V-Veil UP2" consists of a veil and an applicator. A stopwatch is required, which is not included in the box.

### INSTRUCTIONS FOR USE OF "V-Veil UP2"

Wash your hands and wipe them with a clean towel. Open the box and remove "V-Veil UP2" from its packaging by opening it at the dedicated part. Dispose of opened packaging with your normal household waste. Grasp the applicator by the end (in the grip area) with your fingers. Relax and choose the position that feels most comfortable to you: sitting with your legs apart, standing with one leg up or standing with your legs slightly bent. Remove the outer labia and bring the upper part of the "V-Veil UP2" close to the vagina. Gently insert the "V-Veil UP2" veil using the applicator. Continue to push the veil with the applicator until the clamping area remains outside the vagina. The V-shaped end should hang loosely outside the vagina. After inserting the veil remove the applicator from the vagina. To remove the applicator, gently pull it back so that it does not pull the veil. Dispose of the applicator with normal household waste.

**ATTENTION!** Depending on the size of your vagina and how you inserted the intravaginal veil, it is mandatory that the "V" shaped end remains visible so that it can be removed after 2 to 30 minutes.

### TO REMOVE "V-Veil UP2":

Wash your hands and wipe them with a clean towel. Relax and choose the position you find most comfortable from those presented above. The veil can be easily removed, by pulling on the V-shaped end. After removal, place the impregnated veil in a storage container recommended by your doctor or pharmacist.

### RECOMMENDATIONS AND ADVICE:

- Wash your hands before using this medical device.
- Do not use if you have had sexual activity in the last 2 days.
- Do not use if you have washed the vaginal cavity within the last 48 hours.
- If you are pregnant or have blood in the vaginal cavity, use this medical device only on the advice of your doctor.
- The veil should not remain in the vagina for more than 30 minutes, but if you forget to remove it, remove it as soon as you remember.
- Use only one veil at a time.
- The veil should not be reused, even when not impregnated.
- Storage container is not included.
- Use the container recommended by your doctor or pharmacist.
- For more recommendations, talk to your doctor, gynecologist or pharmacist.

### WHO IS PROHIBITED FROM USING THIS PRODUCT?

- Women who:
  - have a fever or flu-like symptoms, even mild ones.
  - have already suffered a toxic shock syndrome and do not have the recommendation of a medical specialist for taking the test.
  - are allergic to plastics and their derivatives.
  - are under 18 years of age, without parental and/or medical consent.
  - are allergic to latex and do not read the instruction sheet.
  - have bleeding and do not have a recommendation from a medical specialist for taking the test.
  - are pregnant and do not have a recommendation from a medical specialist for taking the test.

### TESTS ON "V-Veil UP2":

- Cytotoxicity
- Sensitivity
- Vaginal irritation (for 5 consecutive days)
- Absorption capacity of the veil (1,4 g)

### RAW MATERIAL :

- Veil: 100% Polyethylene/Polyester-bico
- Applicator: 100% Injection molded Polypropylene-Homopolymer for Medical & Pharmaceutical use

### IMPORTANT INFORMATION ON THE TOXIC SHOCK SYNDROME:

Toxic shock syndrome (TSS) is a rare infection that can affect children, men and women (whether they are menstruating or not). It is caused by a toxin produced by a bacterium (staphylococcus) that commonly exists in the body. Half of the cases of TSS recorded to date have occurred in adolescents and women during menstruation. The syndrome has been associated with the use of tampons, but it can also occur when other means of protection are used (pads, liners...). TSS is a potentially fatal infection and its symptoms must be detected very early and treated immediately. Symptoms of TSS may appear suddenly, during menstruation, or shortly afterwards. They may not all be present and may not occur at the same time: sudden fever (>39°C), vomiting, diarrhoea, sore throat, dizziness, fainting, muscle aches, sunburn-like rash (skin may peel a few days later). If you experience these symptoms, you should immediately remove the veil and consult your personal physician or nearest doctor. TSS can occur with or without the use of "V-Veil UP2". If you have already developed a TSS, talk to your doctor before using "V-Veil UP2". By using "V-Veil UP2" correctly and following the recommendations, you will reduce the risk of developing toxic shock syndrome.

### STORAGE:

Store in a clean, dry place away from heat sources.

### INCIDENT REPORTING

**ATTENTION!** Only the instructions for use may be used as a reference for legal information, indications and restrictions. Do not use the product if the packaging is damaged and if the expiry date has passed. If, despite the measures we take in the manufacturing process, you have received a defective product, please do not use it and contact the manufacturer or distributor, clearly indicating the problem and the lot number specified on the packaging. Incidents related to the device can be reported to the competent authority or to a competent authority of the Member State where you are located. The Class I medical device "V-Veil UP2" is manufactured in accordance with the applicable provisions and requirements of REGULATION (EU) 2017/745 and complies with the requirements of EN ISO 13485:2016.

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Copyright V-Veil-UP